
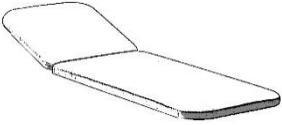


Cleaning recommendations for Novak M examination, therapy and massage tables S

	MATERIALS	CLEANING	DISINFECTION
<p>CONSTRUCTION</p> 	<p>Powder coated steel, chrome plated steel, stainless steel</p>	<p>Regular cleaning is recommended. Use a soft microfiber cloth and a mixture of water and soap. Wipe dry. Use non-abrasive cleaners!</p>	<p>For disinfection, follow the same instructions as for padding.</p>
	<p>Plastic cover</p>	<p>Dust and dry dirt should be removed with soft, damp cloth.</p> <p>Use traditional non-scouring household cleaners Wash well afterwards and clean with a dry cloth.</p> <p>Do not use acetone, chemical cleaning agents, solvents and cleaning agents that are the chemical groups of ketone, ester and aromatic solvents.</p>	
<p>PADDING</p> 	<p>Upholstery: imitation leather</p>	<p>Regular cleaning is recommended.</p> <p>Use warm water and a microfiber cloth or a sponge. In case of stains, use a mixture of soap and warm water. Remove soap residue with damp cloth, wipe with dry cloth and let it dry completely.</p> <p>Blood stains need to be removed immediately with cold water.</p> <p>Massage oil and lotion residues should be removed immediately with a mixture of water and soap.</p> <p>Do not use abrasive cleaners, solvents, acetone or acids. Do not use substances for leather containing oil or grease for maintenance.</p>	<p>Always clean and dry off the padding before disinfection!</p> <p>You can find a list of the recommended disinfectants attached.</p> <p>Always follow the instructions on the packaging.</p> <p>Use only one disinfectant at a time!</p>

Cleaning recommendations for Novak M examination, therapy and massage tables S

Main active ingredients group	Disinfectant
Quaternary Ammonium compound and Amines	Cleanisept AF wipes Incidin Extra Incidin Pro Kohrosolin FF Lysoformin Plus Microbac forte Mikrozid universal wipes Mikrozid sensitive wipes Mikrozid AF wipes Terralin protect
Per-Compounds	Descogen Dismozon Perform
Formaldehyde and/or other Aldehydes resp. Derivates	Incidin Plus Incidin Kohrsolin Optisept
Alcohol	Bacillol AF Bacillol Plus Bacillol 30 foam Bacillol 30 tissues Incidin liquid Meliseptol foam fresh Meliseptol foam pure

Disclaimer

Always follow the instructions for use of each disinfectant. Long-term use cannot exclude the possibility of material deterioration. Novak M does not accept any responsibility with regards to the disinfectant effect of the agents above.

Komenda, April 2023